## Why No touch + forehead thermometer? 為何選擇免接觸額溫槍？

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### or touch forehead 或接觸式

The Braun No touch + forehead thermometer gives you accurate readings instantly. The first ever thermometer with the convenience of two clinically proven options to take the temperature – completely non-invasive no touch or traditional forehead touch. Also good for use on a sleeping child.

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## Breakthrough technology 突破性技術

Innovative technology captures twice as much body heat compared to traditional forehead products.\*

Accuracy sensor guides you to the right distance to take a reading.

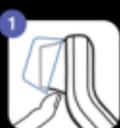
Get the same, professional accuracy reading whether touching your child's forehead or measuring up to 5cm (2 inches) away.

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2 Press power button.  
按下開關按鈕。

3 Position between eyebrows and hold on forehead OR up to 5 cm (2 inches) away from forehead.  
把額溫槍置於眼眉之間、距離前額中心5厘米(2吋)以內位置。

Yellow light will show you where you are aiming.

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NOTE: Backlight remains OFF when below temperature readings are registered:  
注意：當測量到以下體溫範圍，額溫槍背光顯示幕會維持熄滅狀態  
34.4 ≤ T ≤ 35.3°C  
(93.9°F ≤ 96.3°F)

## Temperature taking hints 測量體溫小秘訣

- Patient must be inside for 30 minutes before taking a measurement.  
• 病人必須逗留於室內30分鐘後才可測量體溫。
- For patients measuring their own temperature, it is recommended to use the "touch" option instead of "no touch".  
• 若病人使用本額溫槍為自己測量體溫，我們建議使用“接觸”的方法來測量。

If "0-5cm" appears, move thermometer closer to forehead.  
若顯示幕顯示0-5cm字樣，請將額溫槍移近前額。

**✓ Clear hair from forehead. Position between eyebrows.**  
清除前額的毛髮，把額溫槍對準眉心。

**✓ Hold parallel to forehead. Keep patient's head still.**  
平行對準前額，讓病人保持靜止不動。

**✓ Hold thermometer steady. DO NOT swipe.**  
保持額溫槍靜止不動，切勿移動。

\*A traditional forehead product without any optical system to capture radiated heat

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